

Collection Instructions:

Read all package insert directions carefully before sample collection.
Test results may be invalid if the sample is not collected properly.

***No special diet is required**

- ∅ Place supplied collection paper inside toilet bowl on top of the water.
- ∅ Deposit stool sample on top of the collection paper.
- ∅ Collect sample from the stool before the paper sinks and the stool sample touches water.
- ∅ Flush. The collection paper is biodegradable and will not harm septic systems.

- ∅ Fill in all required information on the sampling bottle.
- ∅ Open the **GREEN** cap by twisting and lifting. **Do not open the white end!**
- ∅ Scrape the surface of the fecal sample with the sample probe. Cover the grooved portion of the sample probe completely with stool.
- ∅ Close the sampling bottle by inserting the sample probe and snap the green cap on tightly. Do not reopen the bottle.
- ∅ Return the sample to the laboratory in the envelope provided. (Do not mail it to Plymedco.)

Container:

